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EU regulatory agent:

US and international patents apply

Specifications are subject to change without notice

SleepSense®

Sensors that make sense
User Manual

Piezo-Crystal Respiration Effort Sensors

Model	Connector/System
1368	Piezo Crystal Effort Sensor - Abdomen / 2 pin safety connector
1369	Piezo Crystal Effort Sensor - Thorax / 2 pin safety connector
1370	Piezo Crystal Effort Sensor - Double Buckle / Safety DIN connectors
1384	Piezo Crystal Effort Sensor - Double Buckle (Sensormedics Ampstar®)
1385	Piezo Crystal Effort Sensor - Double Buckle (Compumedics P Series™)
1387	Piezo Crystal Effort Sensor - Double Loop / Safety DIN connectors
1388	Piezo Crystal Effort Sensor - Double Velcro® tabs / Safety DIN connectors
1392	Piezo Crystal Effort Sensor for Sandman™ Pocket™ - Chest
1393	Piezo Crystal Effort Sensor - Double Loop / PL connector
1395	Piezo Crystal Effort Sensor for Sandman™ Pocket™ - Abdomen
1397	Piezo Crystal Effort Sensor - Double Buckle (Alice 3®)
1398	Piezo Crystal Effort Sensor - Double Buckle (Alice 4®)
1399	Piezo Crystal Effort Sensor - Double Buckle (Alice 5®)

Manufactured by: S.L.P. Ltd.
62 Anilevitz St, Tel Aviv, 67060 ISRAEL
Tel: +972 3 5371281, Fax: +972 3 5371282
www.sleepsense.com

Distributed by: S.L.P. Inc.
1121 E. Main St. #240, St. Charles, IL 60174, USA
Tel: (630) 513 7479, Fax: (630) 513 8478
E-mail: info@sleepsense.com
Toll-free(U.S.): 888-slp-sens (757 7367)

Thank you for purchasing a SleepSense sensor

Description

Piezo crystal respiration effort sensors convert chest or abdominal respiration motions to a small analog voltage that provides a clear, reliable indication of respiration waveforms. These sensors do not require a battery or interface.

Indications for use

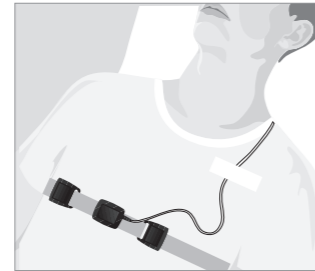
SleepSense® sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Sensor

The patient should lie on his/her back for optimal positioning of the sensor.

To position the sensor:

1. Strap the sensor bands around the patient's abdomen or chest over the nightclothes. Make sure the sensors do not touch the skin.
2. Select the area on the chest/abdomen where the bands will not slide out of place during the night. The sensor box should be slightly off the center line of the body.
*When using double tab sensors, make a small 'wave' of loose band under the sensor.

3. Gently tighten the bands to secure them in place. There will be interference with the signal (noise) if the bands are too tight or too loose.

Tape the cable approximately five inches away from the sensor to reduce strain and enhance reliability. Check that clear, strong signals are being transmitted before leaving the patient.

Cleaning the Sensor

- Wipe the sensor and cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- To sterilize the sensors, use standard gas sterilization procedures.
- Do not autoclave or soak the sensor in disinfectants.

Technical Specifications:

For a complete list of technical specifications please contact us or visit our website

Part types:	Piezo Crystal Sensor
Cable:	White PVC insulation tinsel wire
Sensor:	Small ABS box 1" X 0.7" X 0.5" / Buckles/ Double Loop/ Velcro® Tabs
Recommended filter settings:	High pass: 0.1 Hz / Low pass: 70 Hz, (or as recommended in the system's instructions).
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

Warnings and Precautions

- SleepSense® sensors are for professional use only.
- SleepSense® sensors are for sale by or on the order of a physician only.
- SleepSense® sensors may be used only in conjunction with an approved recording system.
- SleepSense® sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty

SLP warrants the respiration effort sensor to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our Distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our Distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This Warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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