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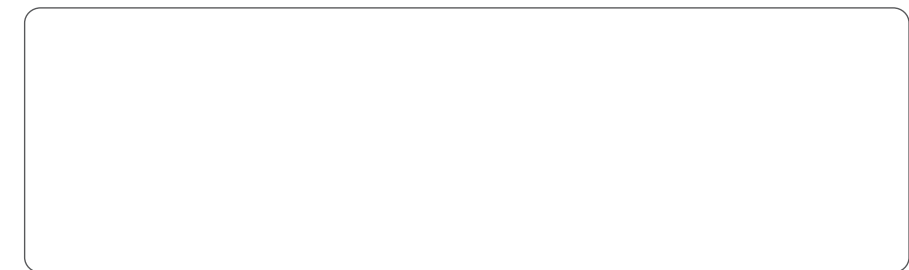
Sensors

English

Thank you for purchasing a SleepSense® sensor.

Description

The SleepSense Pediatric Flow sensor uses three extra small Thermocouples to generate a signal in response to temperature changes. Due to the small size of the Thermocouples, a good signal can be produced by a weaker flow. The output may be connected to any AC (low level) channel on the recorder.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Sensor

The Infant Flow sensor is placed on the bridge of the infant's nose, with the protruding part folded so that the Thermocouples are located under the nostrils.

Positioning the Sensor

1. Bend the sensor's body so that it fits on the bridge of the infant's nose.
2. Apply the sensor on the nose with the three Thermocouples under the nose and the wire towards the forehead. Secure with a piece of facial tape.
3. Carefully bend the base of the three Thermocouples so that the tips are positioned just under the nostrils. Be sure that they do not touch the nose.
4. Draw the wire over the infant's forehead and tape to enhance reliability.
5. Check that clear, strong signals are being transmitted before leaving the patient.

Cleaning the Sensor

- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- Do not autoclave or soak the Interface Cable in disinfectants.

Technical Specifications:

For a complete list of technical specifications please contact us or visit our website.

Sensor:	3 miniature thermocouple elements
Recommended filter settings:	High pass: 0.1 Hz / Low pass: 70 Hz
Signal Output:	Approximately 0.2 mV for normal respiration
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

Warnings and Precautions

- SleepSense sensors are for professional use only.
- **Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors may be used only in conjunction with an approved recording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

Limited Warranty
SLP warrants the Infant Thermal Sensor to be free of defects in materials and workmanship for a period of six months from the date purchased. The sole liability of SLP and our distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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