Disposable Flow Sensors

Italiano

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Disposable Flow Sensors

English

Thank you for purchasing a SleepSense® sensor.

Description

The interface cable is used to connect the 1426/1427/1428 disposable thermocouple respiration flow sensors to the PSG system. Internal battery and filtering circuits provide clean and reliable respiration signals to the system. Battery should last at least two years of continuous use. The output may be connected to any AC (low level) channel on the recorder.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Sensor

The respiration flow sensor is placed on the patient's upper lip, under the nostrils.

Positioning the Sensor

- Slide the sensor's connector (square end) into the cable's connector slot. Hold the sensor very close to the connector and apply even, steady pressure to insert. Once connected, the sensor should not be disconnected unnecessarily, as this may break the sensor.
 - Bend the two nasal prongs forward to prevent them from entering the nostrils. Press hard to make the prongs stay in place.
 - Align the tip of the oral prong at the center of the mouth/upper lip.
 - Fix the sensor in place using pieces of skin compatible adhesive tape.
 - Draw the wire over the patient's ear and tape the wire on the cheek and on the neck to reduce strain and enhance reliability.
- Check that clear, strong signals are being transmitted before leaving the patient.

Cleaning the Sensor

- Wipe the sensor and cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- To sterilize the sensors, use standard gas sterilization procedures.
- Do not autoclave or soak the sensor in disinfectants.

Technical Specifications:

For a complete list of technical specifications please contact us or visit our website.

Cable:	White PVC insulation lead wire
Sensor:	Disposable Flow Sensor # 1426/ 1427/ 1428
Recommended Filter Settings:	High Pass: 0.1 Hz/ Low Pass: 70 Hz
Signal Output:	Approximately 0.2 mV for Normal Respiration
Operating conditions:	5°C (40°F) - 40°C (104°F)
Storage temperature:	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity:	5% - 95% (Non-condensing)

Warnings and Precautions

- SleepSense sensors are for professional use only.
 - Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
 - SleepSense sensors may be used only in conjunction with an approved recording system.
 - SleepSense sensors are only intended for use on healthy skin.
 - Discontinue use if sensor shows signs of wear, damage, or exposed metal.
- Attention:** contains a lithium battery - dispose of properly.

Limited Warranty

SLP warrants the Infant Thermal Sensor to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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