Italiano

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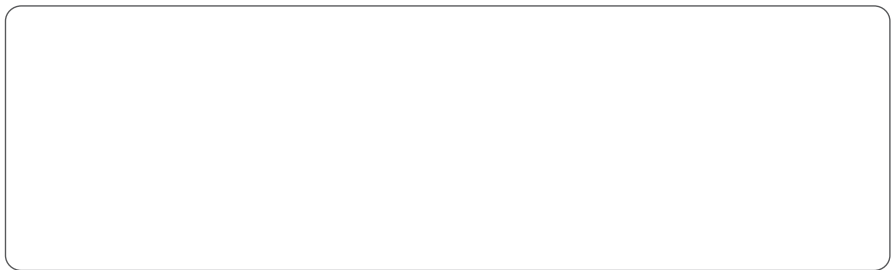
Specifications are subject to change without notice

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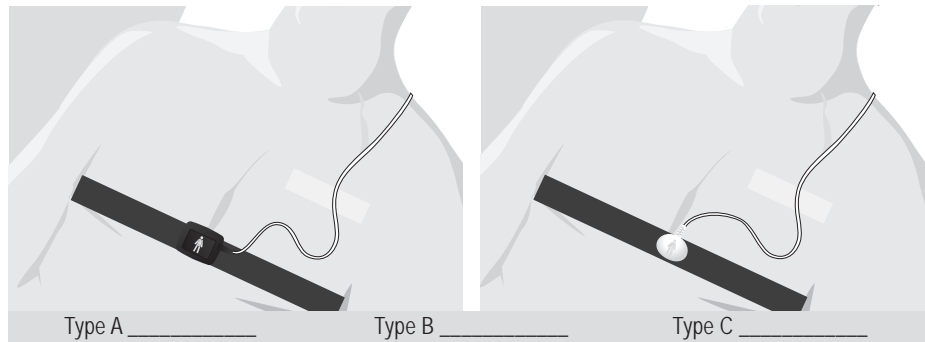
Position Sensors

English

Thank you for purchasing a SleepSense® sensor.

Description

Using gravity referenced switches, body position sensors provide a signal that indicates to the system the patient's sleeping posture (supine, prone, left, right or sitting). These sensors should be connected to the appropriate input in the system.

Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

Applying the Sensor

The patient should lie on his/her back for optimal positioning of the sensor.

Positioning the Sensor

- Place the sensor belt around the patient's chest over the night clothes or apply the position sensor on the effort sensor belt. Locate the position sensor at the exact center of the body over the sternum, and secure in place.
 - Carefully tighten the body position belt to secure the sensor in place. A loose belt may rotate and produce error in indication during the night.
- Tape the cable about five inches away from the sensor to reduce strain and enhance reliability.
 - Connect the output connector to the appropriate input on the system.
 - Check that the positions of the body are correctly indicated before leaving the patient.

Cleaning the Sensor

- Wipe the sensor and cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- To sterilize the sensors, use standard gas sterilization procedures.
- Do not autoclave or soak the sensor in disinfectants.

Technical Specifications:

For a complete list of technical specifications please contact us or visit our website.

Item:	Type A	Type B	Type C
Signal output:	5 DC levels	8 resistance values	5 resistance values
Power:	Battery, non-replaceable	Powered by the system	
Part type:	Position Sensor		
Operating conditions:	5°C (40°F) - 40°C (104°F)		
Storage temperature:	-20°C (-4°F) - 60°C (140°F)		
Operating and storage humidity:	5% - 95% (Non-condensing)		

!	Warnings and Precautions
	<ul style="list-style-type: none">SleepSense sensors are for professional use only.Caution: US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.SleepSense sensors may be used only in conjunction with an approved recording system.SleepSense sensors are only intended for use on healthy skin.Discontinue use if sensor shows signs of wear, damage, or exposed metal. <p>Attention: certain sensors contain a lithium battery - dispose of properly.</p>

Limited Warranty
SLP warrants the body position sensor to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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