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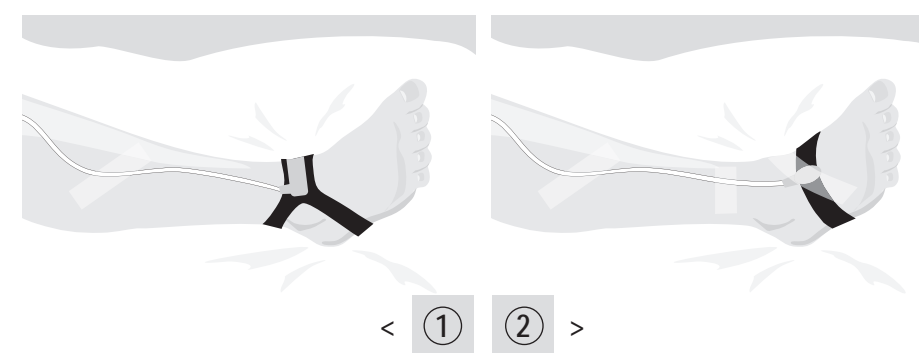
Thank you for purchasing a SleepSense® sensor.

### Description

The limb movement sensor is placed on the back of the hand or on the ankle or foot (depending on model) and produces a signal that is directly proportional to the hand or leg movement.

### Indications for use

SleepSense sleep-lab sensors provide a qualitative measure of sounds, respiratory-effort, flow, body position or limb movements, for recording on an approved data acquisition system. They are intended for use on children and adult patients who are screened during sleep disorder studies at a sleep laboratory or the patient's home.

### Applying the Sensor

The patient should lie on his/her back for optimal positioning of the sensor.

### Positioning the Sensor

#### Piezo sensor (picture 1)

- Place the limb movement sensor on the ankle-band or on the wrist. When using on the ankle, place on the front of the foot over the tendons. **It is recommended to use the SleepSense ankle-band, (part No.1730), over the heel.** For use on the wrist, place on the inside of the wrist, over the tendons.
- Position the sensor so that it will not move during the night.

#### Switch sensor (picture 2)

- Apply on the back of the non-dominant hand, or on the foot, using a piece of tape. **It is recommended to use the SleepSense ankle-band, (part No.1343), over the heel.**

- Tape the cable about six inches away from the sensor to reduce strain and enhance reliability.
- Check that clear, strong signals are being transmitted before leaving the patient.

### Cleaning the Sensor

- Wipe the sensor and cable with a non-corrosive (to plastic) cleanser to clean before use.
- Make sure the complete sensor assembly is thoroughly dry before reusing it.
- To sterilize the sensors, use standard gas sterilization procedures.
- Do not autoclave or soak the sensor in disinfectants.

### Technical Specifications:

For a complete list of technical specifications please contact us or visit our website.

Item:	Type A Piezo	Type B Switch
Sensor:	Gray silicone cover / Velcro® hook back	Gray TPE cover
Signal output:	Approximately 1 mV for small movements	Switch momentarily opens when moved
Intended for system:	Any system that accepts AC Inputs	Alice 5
Operating conditions:	5°C (40°F) - 40°C (104°F)	
Storage temperature:	-20°C (-4°F) - 60°C (140°F)	
Operating and storage humidity:	5% - 95% (Non-condensing)	

**! Warnings and Precautions**

- SleepSense sensors are for professional use only.
- Caution:** US Federal Law restricts this device to sale, distribution and use by or on the order of a physician.
- SleepSense sensors may be used only in conjunction with an approved reording system.
- SleepSense sensors are only intended for use on healthy skin.
- Discontinue use if sensor shows signs of wear, damage, or exposed metal.

**Limited Warranty**  
 SLP warrants the limb movement sensor to be free of defects in materials and workmanship for a period of twelve months from the date purchased. The sole liability of SLP and our distributor(s) is limited to replacement or repair of the product at the option of SLP with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall SLP or our distributor(s) be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended only for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect, or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

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