
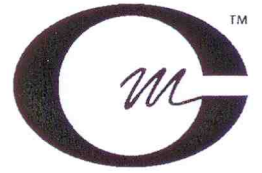


Directions for use

- > For use only with Compumedics Grael, part number 8028-0001-01.
- > Tape the microphone to the patients neck using tape intended for use on the face.
- > Clean with a solution of warm water and mild liquid soap.
- > If disinfecting is required use either a 0.5% solution of Cidex OPA or a solution of diluted bleach.
- > Refer to the Grael User Guide, Compumedics part number 0228-0001-01, for further details.

Grael Microphone

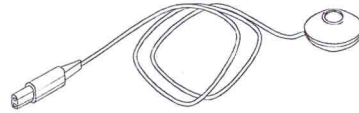

COMPUMEDICS
LIMITED

A.C.N. 006 854 897
30 – 40 Flockhart St
Abbotsford, VIC., 3067
AUSTRALIA
Tel: +61 3 8420 7300
Fax: +61 3 8420 7399



Caution: U.S. federal law restricts this device to sale by or on the order of a physician.

Part Number: 7028-0002-01